


Pulsed Frequency™

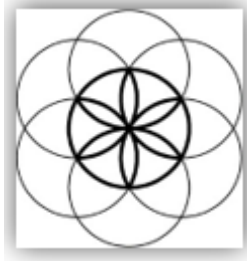

Joint PEMF Basic Software
Download, Installation and
User Guide

Contents


Account Registration.....	3
Create an Account.....	4
Download the Software	6
Chrome.....	7
Edge.....	8
Firefox	10
Install the Software.....	12
Just Feel Better!	17
Connect JOINT PEMF to Your Computer.....	19
When the Software is Run	21
Load a Program into the Joint PEMF.....	23
Troubleshooting.....	24
Joint PEMF Manual Detection.....	27

Account Registration

To download the Pulsed Frequency software, please go to www.pulsedfrequency.com.



Click where the arrow is pointing. You will be taken to a Login screen.

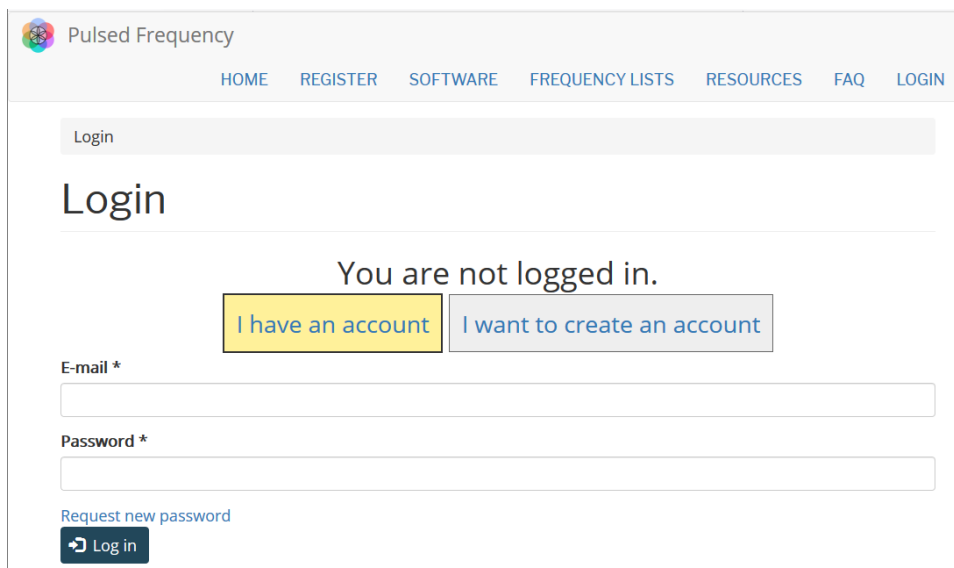

If you already have an account, click **I have an account**. Enter your email address and password and then click **Log in**.

Create an Account

If you do not have an account, select **I want to create an account**.

Pulsed Frequency

HOME REGISTER SOFTWARE FREQUENCY LISTS RESOURCES FAQ LOGIN

Login

Login

You are not logged in.

I have an account I want to create an account

Please check your spam folder for the confirmation email.

E-mail *

Create new account

Enter your email address, then click **Create new account**. Further instructions will then be sent to your email address. The confirmation email will look something like this.

From: support@pulsedfrequency.com
 To: rod@server.com
 Cc:
 Subject: Account details for rod_564 at Pulsed Frequency

Dear Registered User,

Thank you for registering at Pulsed Frequency. You may now log in by clicking this link or copying and pasting it to your browser:

<https://pulsedfrequency.com/user/reset/564/1534971011/JKZCwxsqdPQhoLdXsoli2wnP8Z554uiCxLi-LJuTjH0>

This link can only be used once to log in and will lead you to a page where you can set your password.

After setting your password, you will be able to log in at <https://pulsedfrequency.com/user> in the future using:

username: rod@server.com
 password: Your password

-- Pulsed Frequency team

Click on the link in the email and you will be taken back to the website to reset your password.

You have just used your one-time login link. It is no longer necessary to use this link to log in. Please change your password. ×

[View](#) [Edit](#) [Orders](#)

E-mail address *

Password

Confirm password

Email updates
 Subscribe

Locale settings
Time zone

Enter a New password in both fields.

You have just used your one-time login link. It is no longer necessary to use this link to log in. Please change your password. ×

[View](#) [Edit](#) [Orders](#)

E-mail address *

Password
 ✓

Confirm password
 ✓

Email updates
 Subscribe

Locale settings
Time zone

After you have changed your password, click the **Save** button. You will be told that the changes have been saved.

Download the Software

If you aren't already logged in, log into your account, then click the Software menu item.

The screenshot shows the Pulsed Frequency website header with navigation links: HOME, SOFTWARE, FREQUENCY LISTS, RESOURCES, FAQ, and LOGOUT. A dropdown menu is open under the SOFTWARE link, showing options: DOWNLOAD, INSTALL / USER GUIDE, VIDEO GUIDE, PRO RESOURCES, PRO, BASIC, EM27 BASIC, EM272 BASIC, JOINT PEMF BASIC (highlighted with a red arrow), MR7 BASIC, and PLASMA BASIC. Below the menu, a banner features the text "Facilitating a New Paradigm" and "Building Software programs that generate:" followed by a list of field types: Pulsed Magnetic Fields generated by Magnetic Inductor Coils, Pulsed Light Fields generated by LED Diodes, Pulsed Microcurrent Fields generated by Skin Electrodes, and Pulsed Electromagnetic Fields generated by Plasma Fields. The background of the banner shows a red lotus flower on water.

Point to **Download**, point to **Basic**, then click **Joint PEMF Basic**. When you do this, you will be taken to the following page.

The screenshot shows the software download page for "WAVEOMICS™ Software: Pulsed Frequency™ Joint PEMF Basic". The breadcrumb trail is "Software / Download / Basic / JOINT PEMF BASIC". Below the title, there are tabs for "View", "Edit", "Revisions", "Access control", and "Devel". A message states: "You must have an account and be logged in to download Waveomics™ software." Below this message is a dark blue "Download" button.


Click the **Download** button.

What happens next depends on the web browser you are using. Here are instructions specific to the three main browsers being used today.

Chrome




When you click the **Download** button, a notification displays in the lower left hand corner of the browser that alerts you to the file being downloaded. You will see the number of bytes that have been downloaded and the total size of the file.

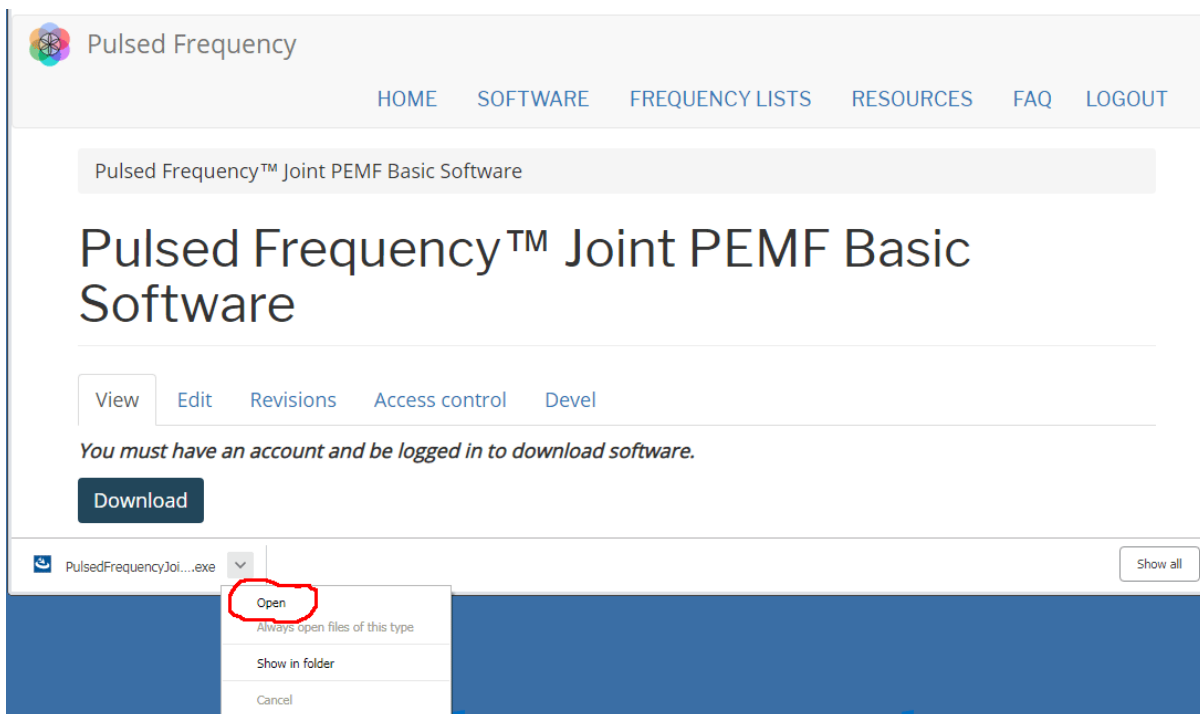


When the file download has completed, click the down arrow and then click **Open**. This will run the installation program for the **JOINT PEMF Basic** software.

Edge


When you click the **Download** button, a small rectangular window will display at the bottom of the screen.

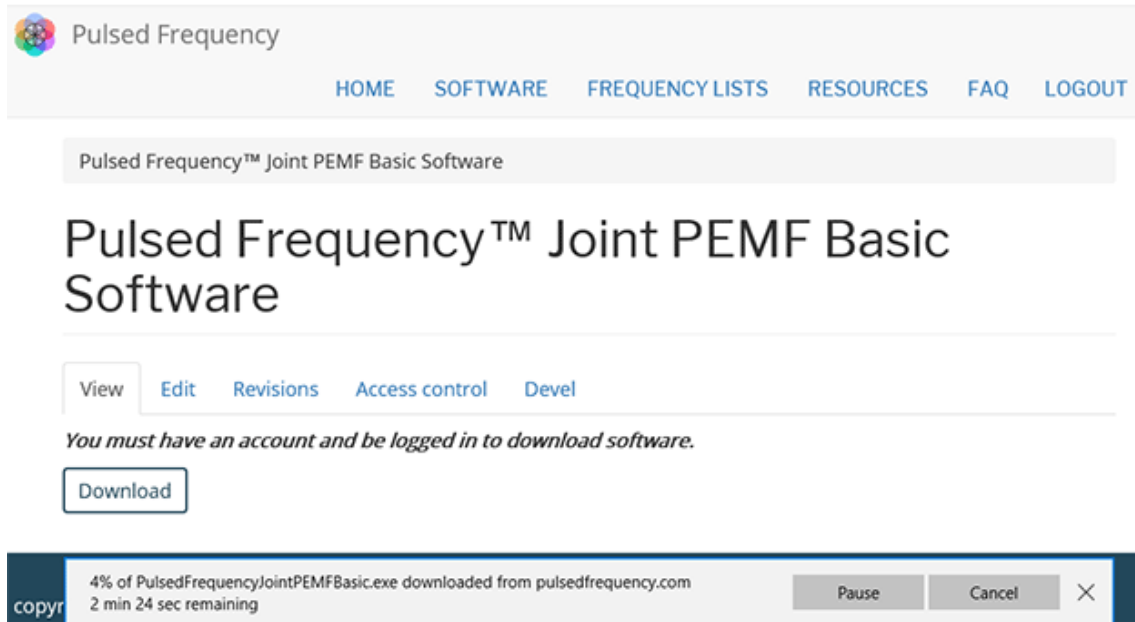


This window will show the progress of the file being downloaded.

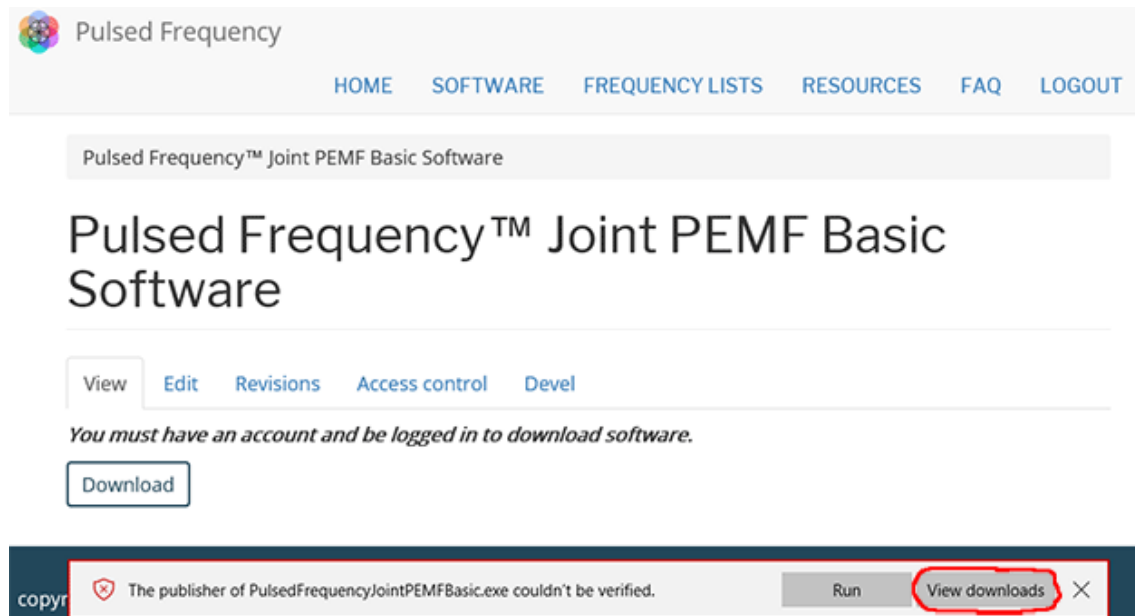

When file download has completed, click the **View Downloads** button.

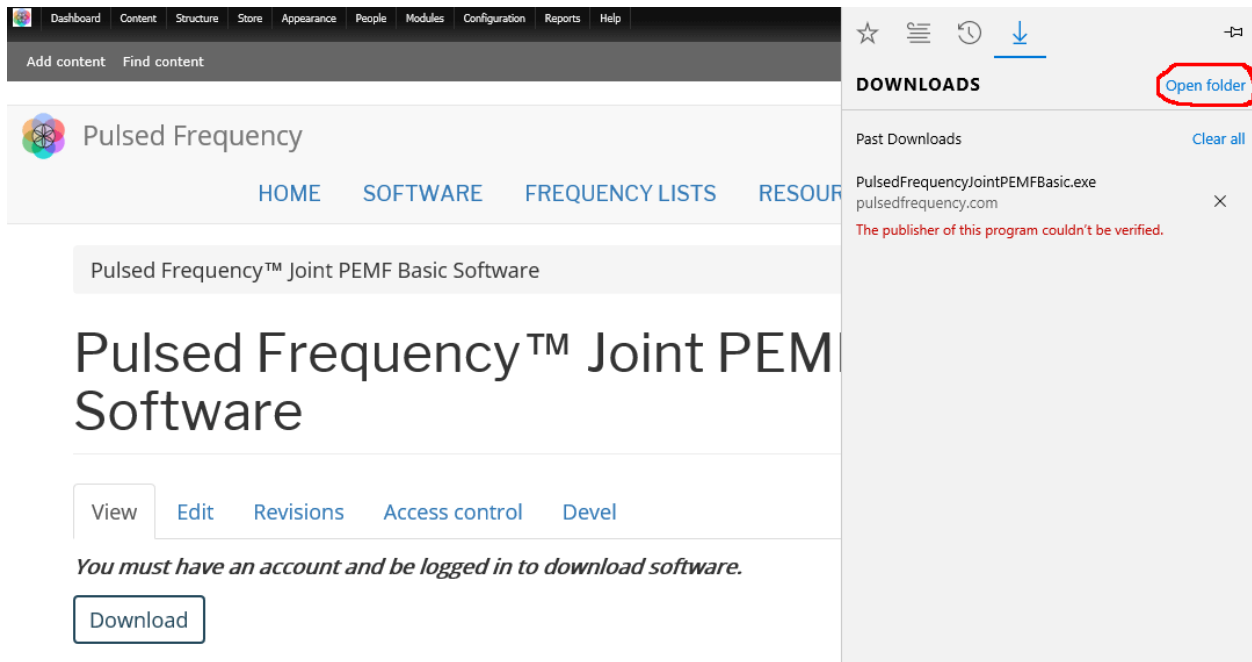


Next, in the upper right hand corner of the screen, click **Open Folder**.

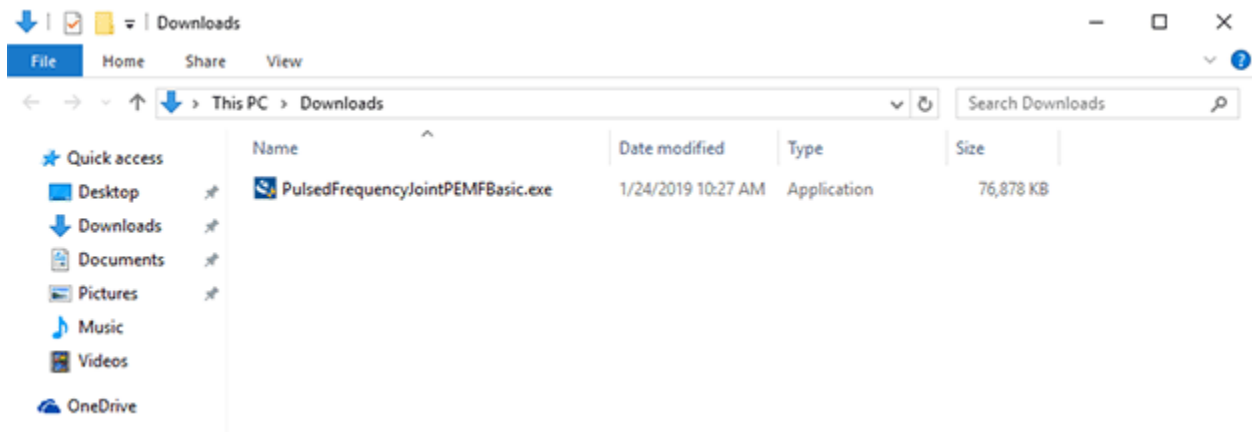


Finally, double-click **PulsedFrequencyJointPEMFBasic.exe** to run the installation program for the **JOINT PEMF Basic** software.

Firefox


When you clicked the **Download** button, the following dialog box popped up.

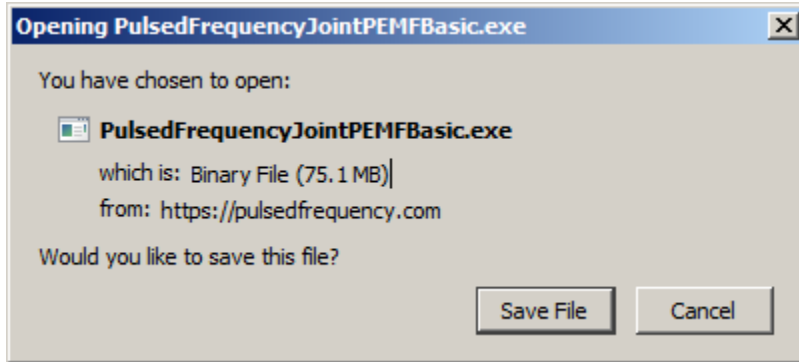


In this dialog box, click the **Save File** button.

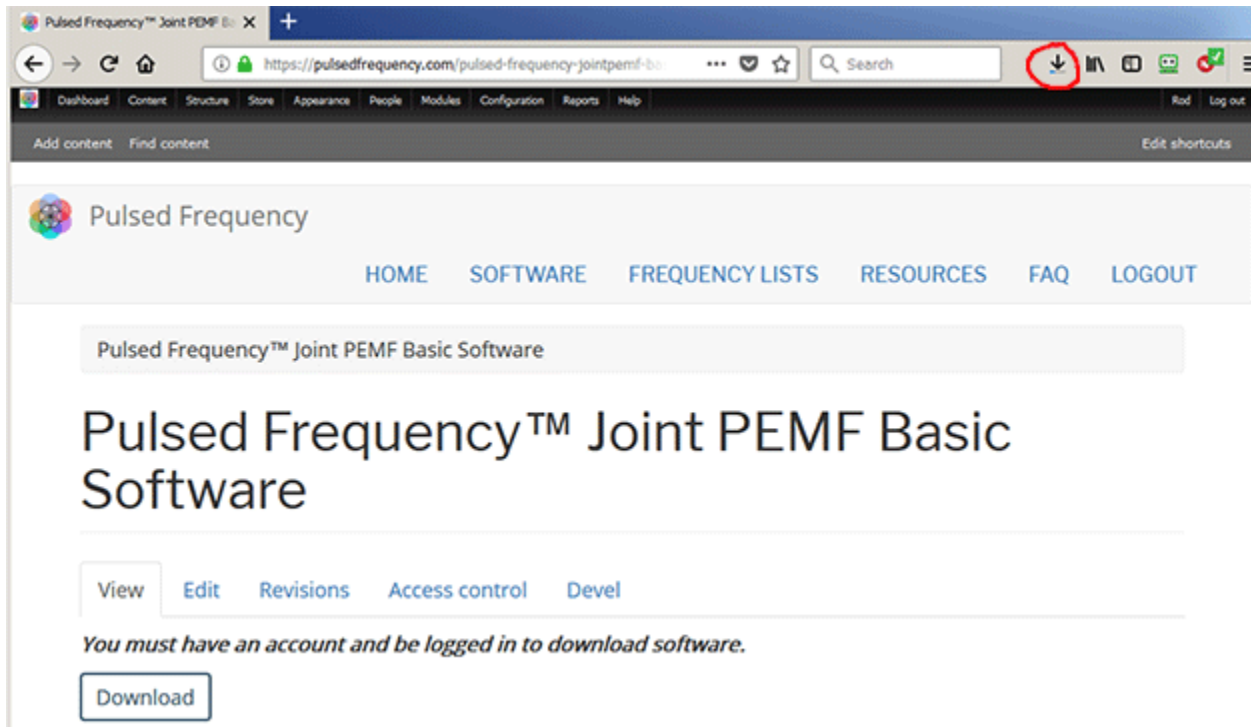

Next, go to the upper right hand corner of the browser and click the **Down Arrow**.

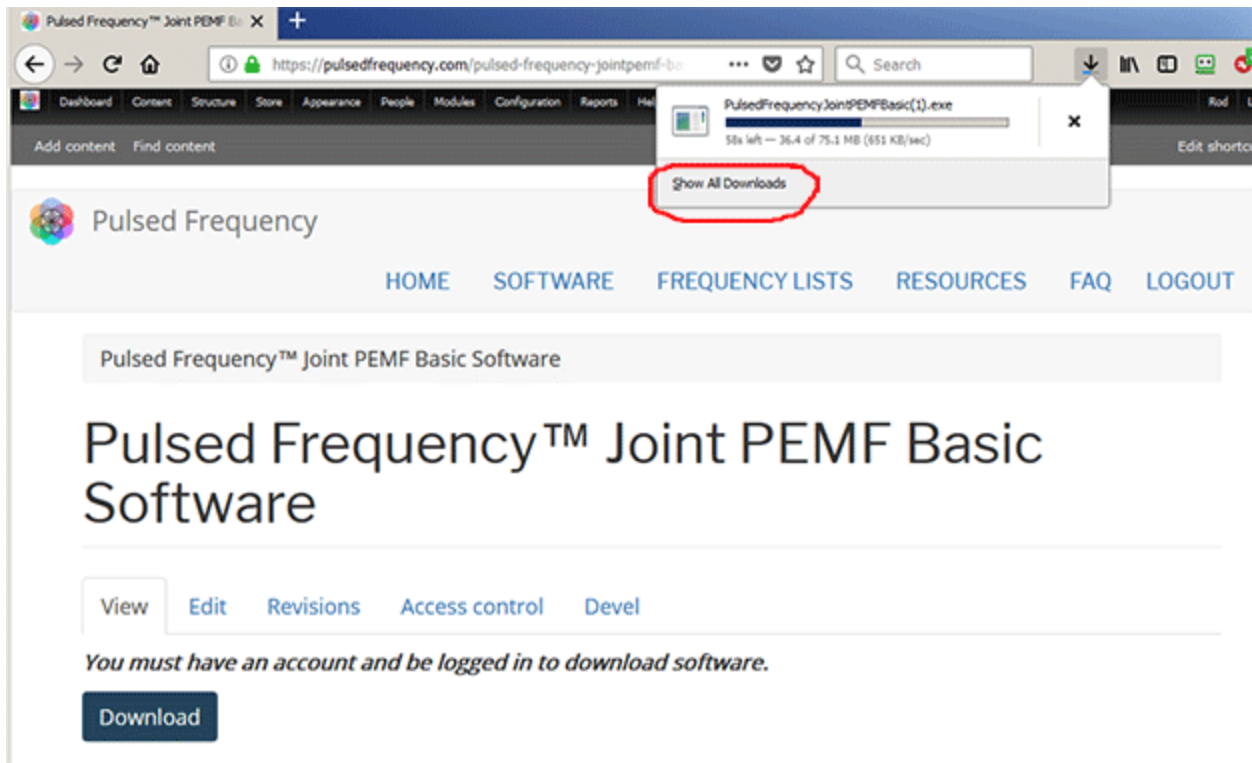


A small window will appear. Click the link in the lower left that says, **Show All Downloads**.

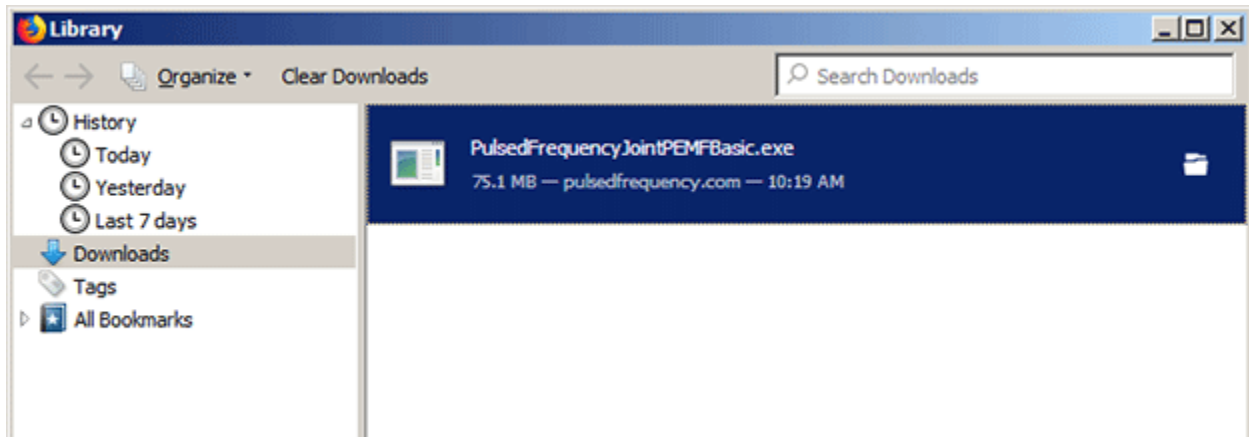


Next, the above window will pop up where you can watch the progress of the downloading file. When the file has downloaded, double-click on **PulsedFrequencyJointPEMFBasic.exe** and the installation program will run.

Install the Software


To install the JOINT PEMF Basic software, double-click on the downloaded file.

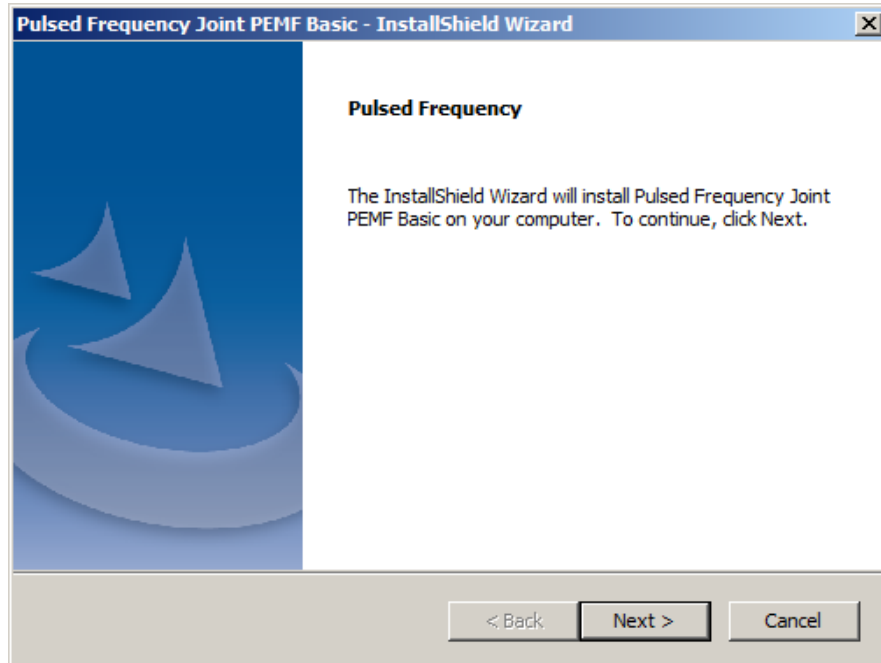


On this Welcome screen, click the **Next** button.

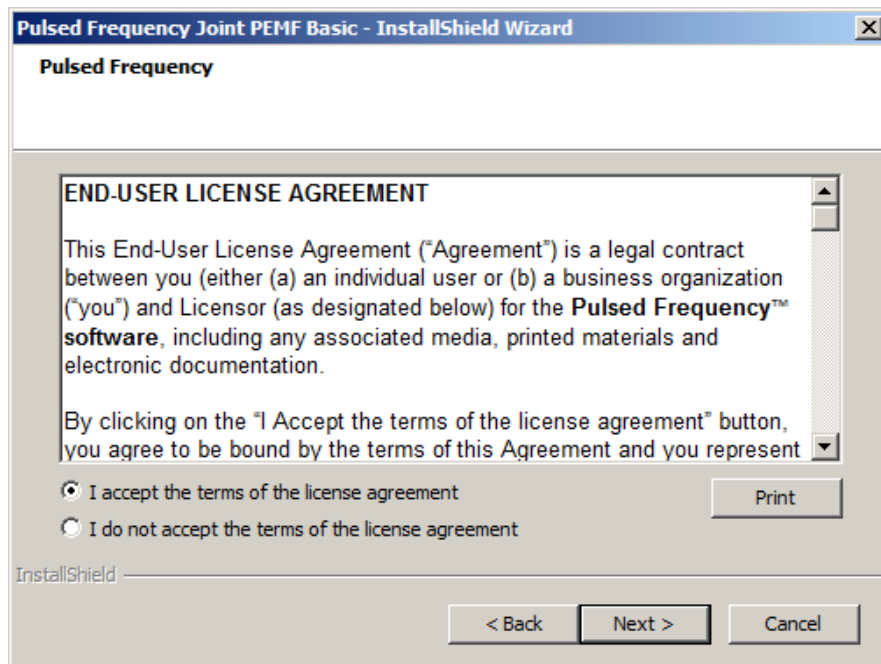


Click **I accept the terms of the license agreement**, then click the **Next** button.

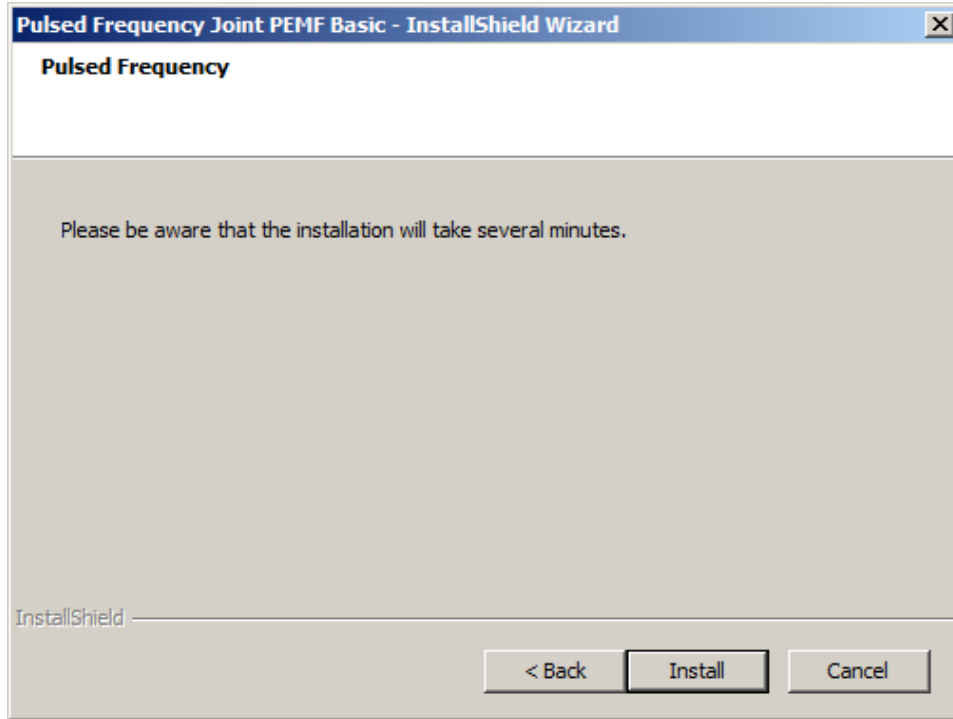


To begin the software installation, click the **Install** button.

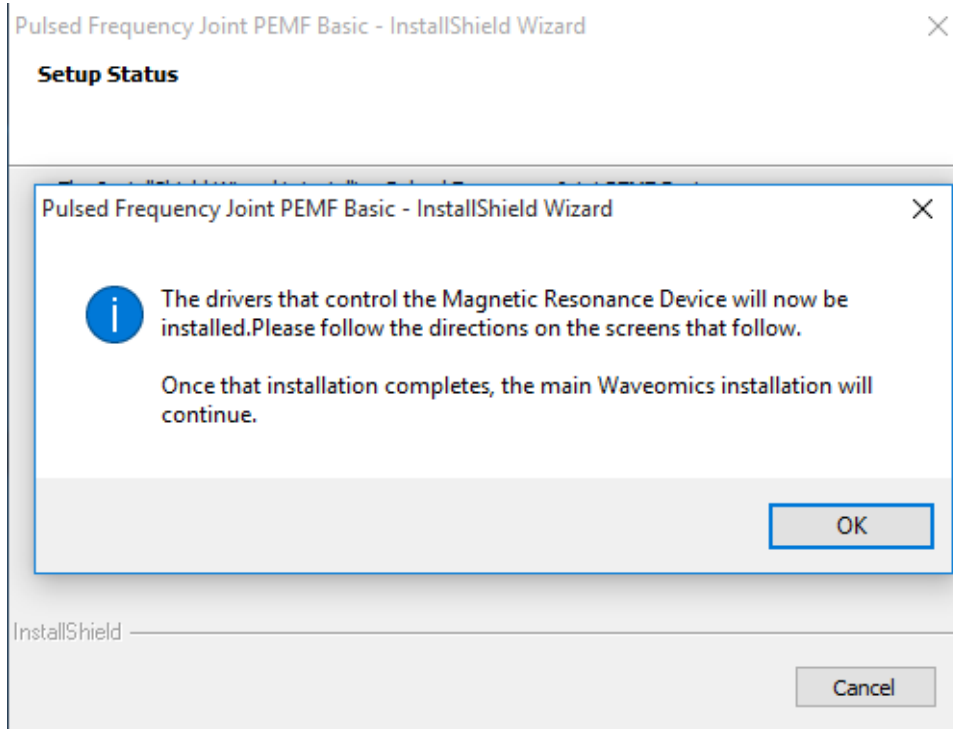


At this point, this dialog box will appear and inform you that the drivers needed for your JOINT PEMF device will now be installed. Click the **OK** button to begin.

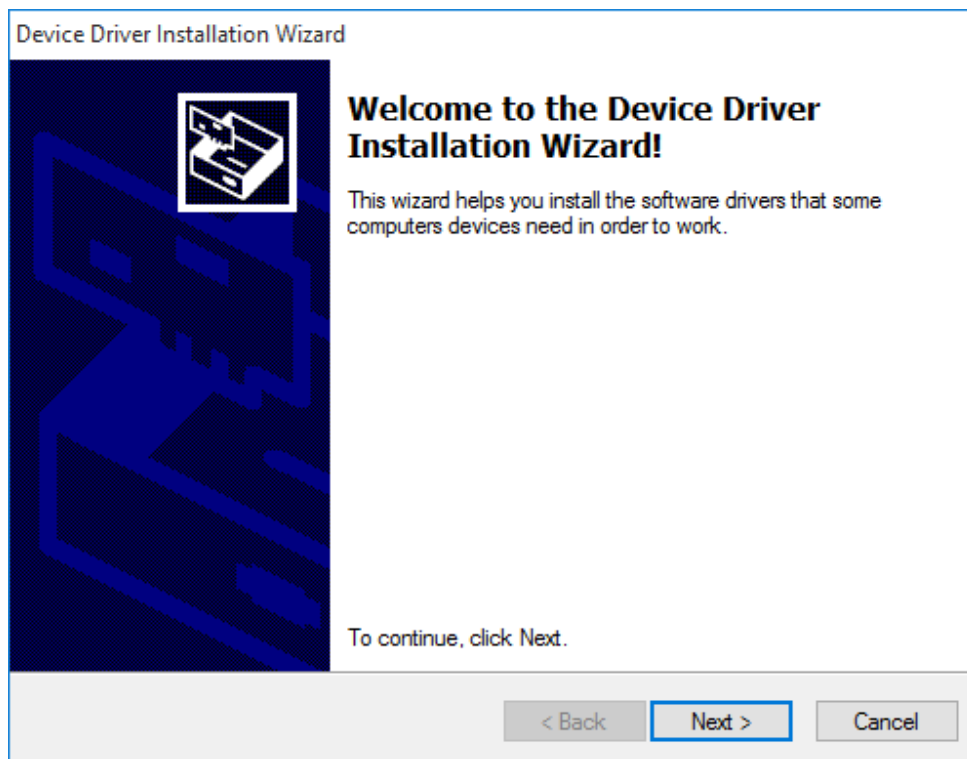


On this screen click the **Next** button.

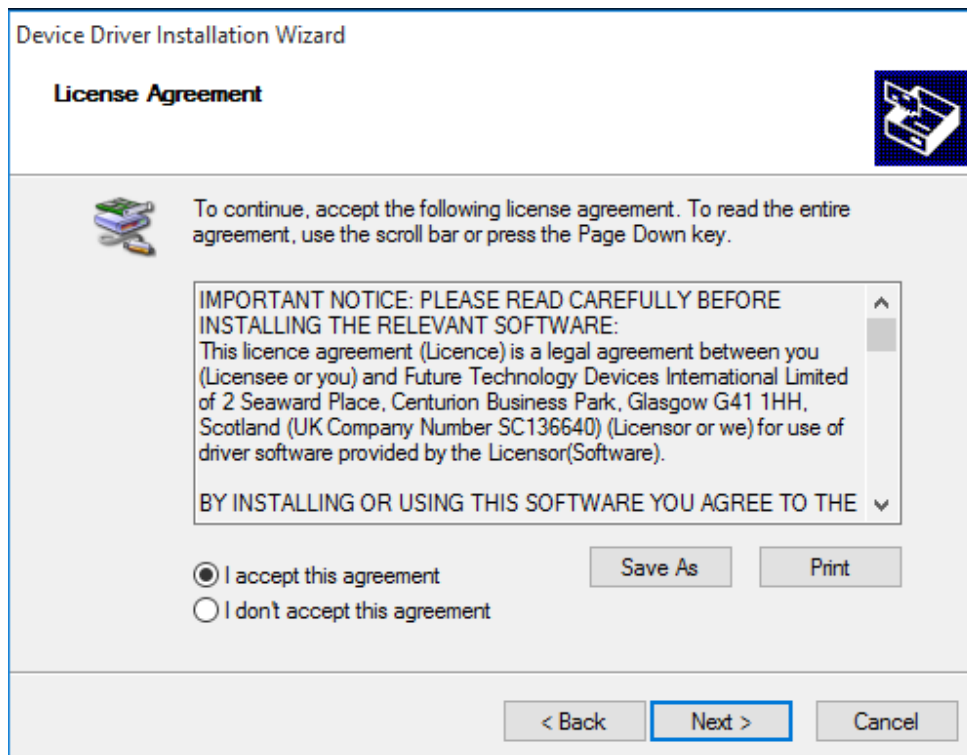


Accept the license agreement, then click the **Next** button.

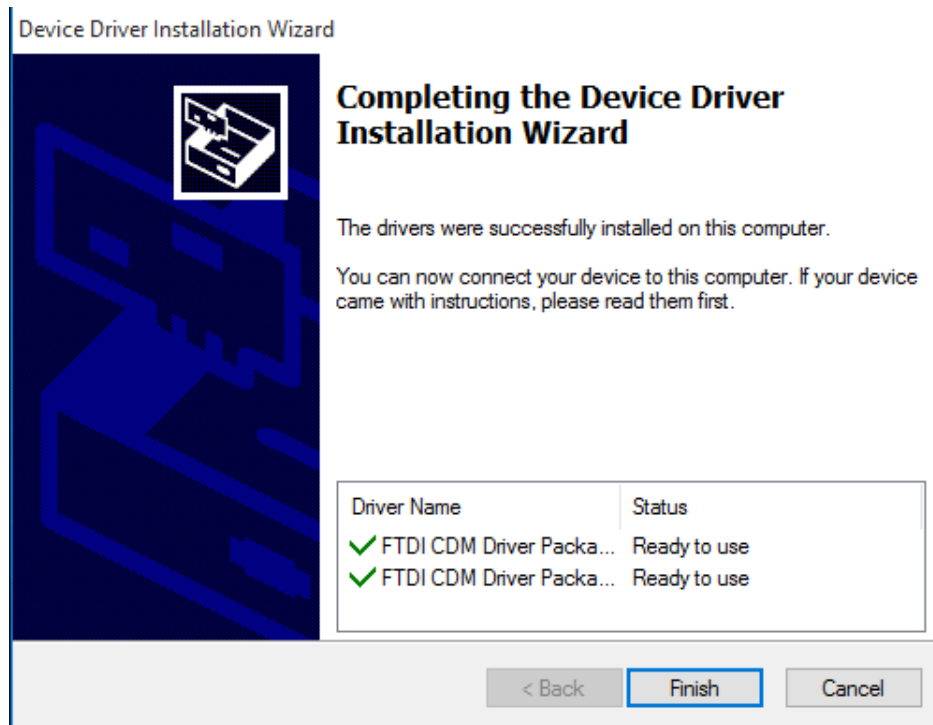


Click the **Finish** button to complete the driver installation.

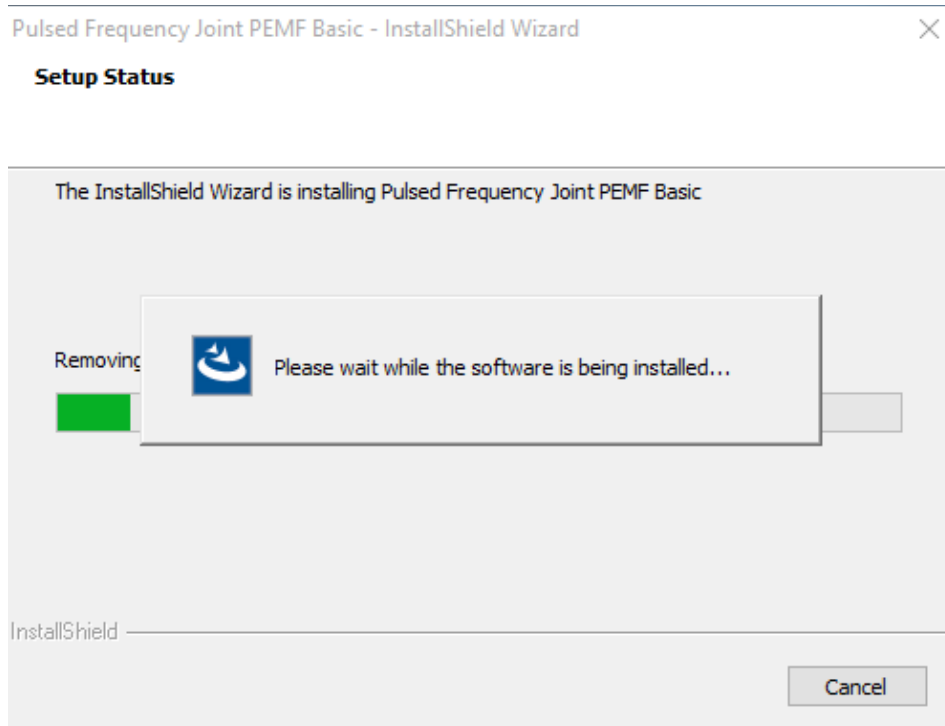

At this point, the JOINT PEMF Basic software will be installed. Just wait until it has completed.

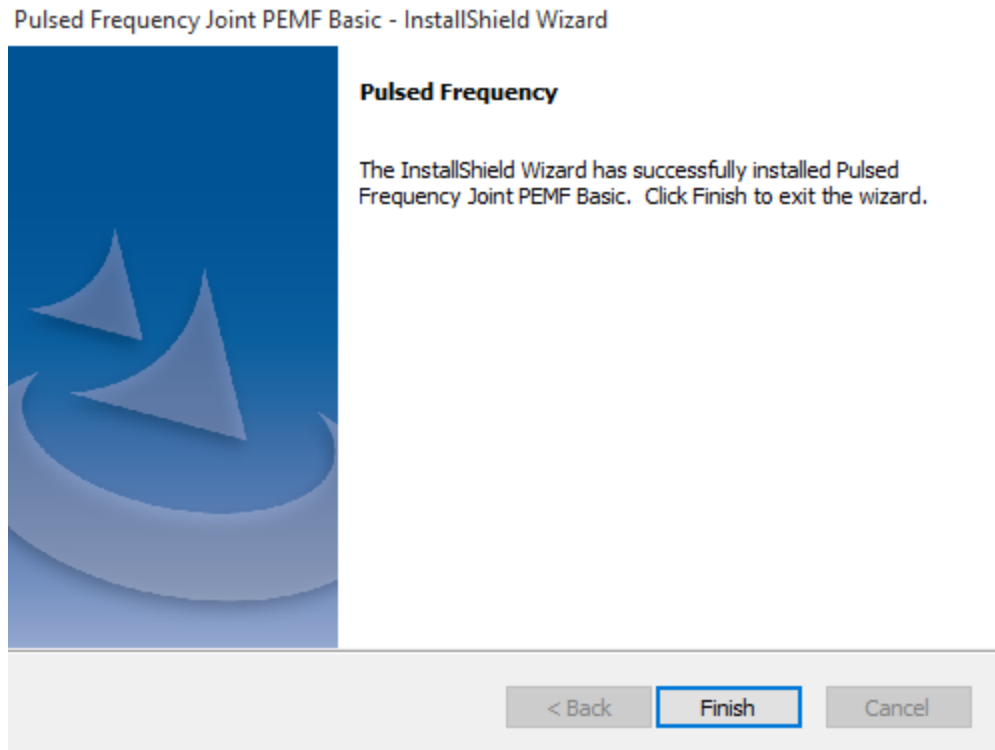

When you see this screen, the installation is complete. Click the **Finish** button.

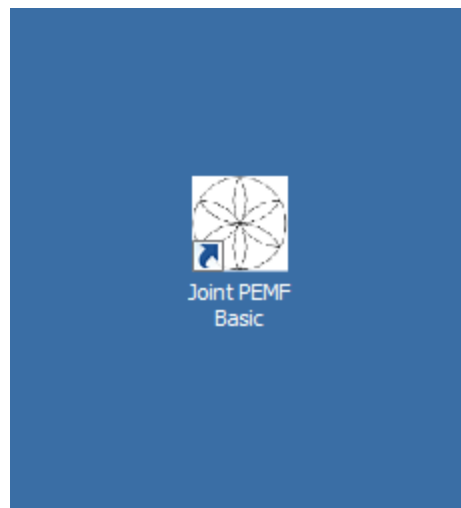

During installation, a shortcut to the JOINT PEMF Basic software was placed on your desktop. Double-click this shortcut to run the software.

Just Feel Better!

Joint PEMF LED and Magnetic Pulsed Frequency Generator

The following steps are instructions for the Joint PEMF Pulsed Frequency Generator. The device comes with a Frequency generator box, 2 LED pulsing light disks and/or 2 pulsing magnetic disks, and a 110 volt charger / power cord.

Setting up the Joint PEMF Device

Insert the charger cable into the round port to the left of the On/Off button and the other end into a 110 volt electrical outlet. This will charge the device. A full charge will take approximately 4-8 hours. The light surrounding the On/Off button will flash Red/ Green when the charge is complete. The JOINT PEMF will run when the charger is connected or disconnected from the device. The Joint PEMF device will run approximately 10 hours on battery power once charged.



To turn the Joint PEMF device ON, Push the large On/Off button in the middle, until the lighted ring turns green. This ring will stay green while the programs are running. The Green On/Off light will turn Flashing Red when the device needs a charge. To turn the Joint PEMF device OFF, press and hold the On/Off button for about two seconds. The device will beep and turn off. If running programs when the charger is plugged in, the green light will flash.



The JOINT PEMF offers the option of using Magnetic disks or LED light disks. If using the magnetic disks, it is suggested that you face the flower label side away from the body, so that you can see the flashing green lights on the flower. The magnetic pulses will go through clothing, so there is no need to have them against the skin. unlike the LED lights.



The Magnetic disks or the LED disks plug into the RJ45 jacks on the back side of the JOINT PEMF. Do not wrap the disc cables or Power cable around the JOINTPEMF device. Instead, unplug them and wrap each cable individually.



If using the two red LED light disks, plug them into the RJ45 plug jacks on the bottom of the JOINT PEMF. When using the LED lights, the lights MUST face the skin. The lights will not penetrate through cloth. The LED lights will feel warm. The lights need to breathe. Do not sandwich them between you and furniture. If the LED lights get too warm for your skin, move to another position. LEDs can be held against the skin with a brace, sock or ace bandage.



The Joint PEMF device has the ability to be reprogrammed. To the right of the On/Off switch, there is a mini USB port. You will need to use a USB/ Mini USB cable to connect the unit to your computer if you want to transfer programs. Programs and the appropriate software can be obtained from www.Pulsedfrequency.com.



You may find it convenient to use the LEDs against your body during the day, then use the Magnetic disk under your fitted sheet at night, so that you can sleep on top of them. This regimen will maximize the treatment effect. You may also tuck the Joint PEMF device and the two magnetic disks into a brace around a knee or other joint.

Joint PEMF: Useful Tips

It's very important to drink a minimum of 32 ounces of purified water before a treatment, and another 32 ounces of purified water after the treatment. The water helps the frequencies flow through the body and cleans out the debris that the cells are casting off.




You may experience a slight dizziness or detoxing flu-like effects from this treatment. If this happens, reduce the treatment time. Slowly increase daily treatment time until all programs are run and the unit shuts off. Also increase your water intake until symptoms subside.

It is suggested not to use electronic devices while being treated, such as computers, cell phones, I-pads and Kindles. Electronic devices may not work within the electro-magnetic field. This caveat does not apply to LED Lights.

Disclaimer: The JOINT PEMF device does not claim to treat or correct any medical diseases, injuries or symptoms. The JOINT PEMF device is considered to be a Wellness device.

Connect JOINT PEMF to Your Computer

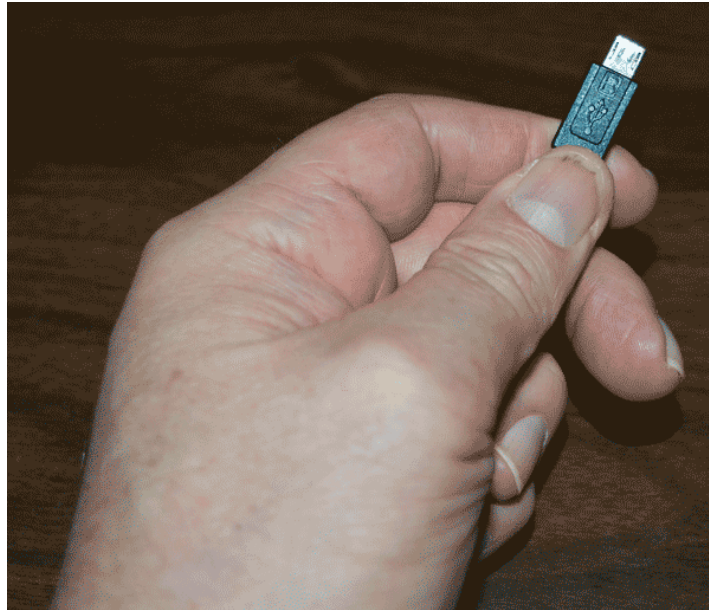


To connect the JOINT PEMF to the computer, take the supplied Mini-USB cable and orient the smaller end so that it can be plugged into the device.



Make sure the JOINT PEMF is OFF and plug the Mini-USB cable into the mini-USB jack in the unit. The JOINT PEMF is OFF if the green LED ring light is not lit.




Plug the larger end of the USB cable into your computer.

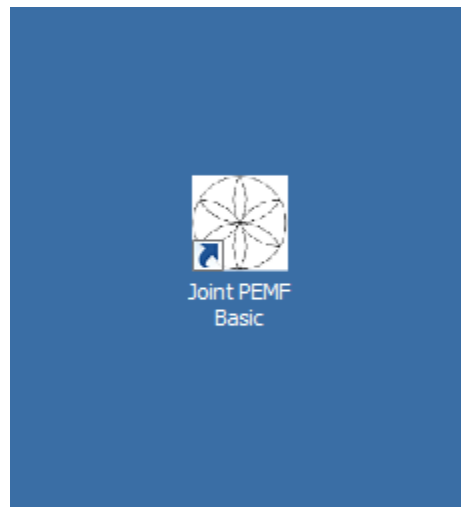


To run the JOINT PEMF Basic software, double-click the **Joint PEMF** Basic shortcut on the desktop.

When the Software is Run


When you run the software, it will first check to see if a software update is available.

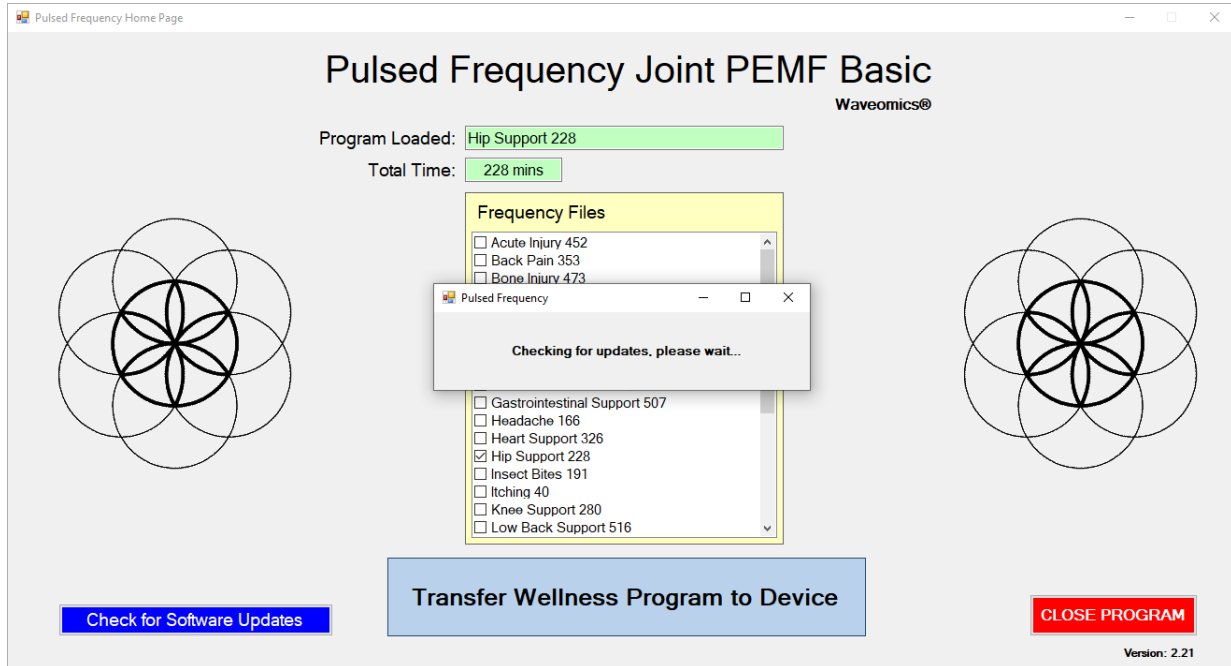


During this process, a small dialog box will display. If no update is available, the dialog will just go away.

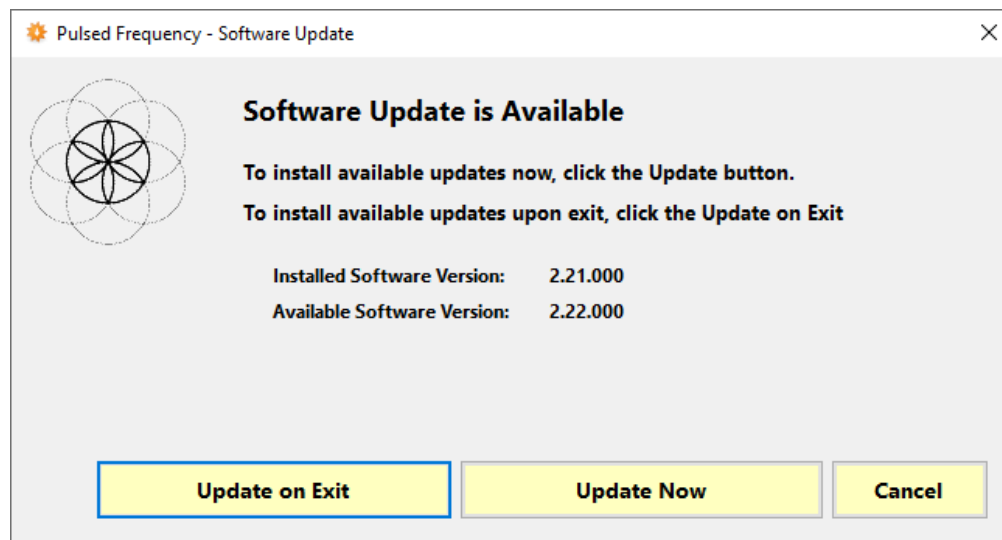

If an update is available, the above dialog box will appear. If you click **Update on Exit**, this box will disappear and you will be back at the main page. The software will then be updated when you click the **Close Program** button. If you click **Update Now**, the software will be updated immediately. If you click **Cancel**, no update will be performed.

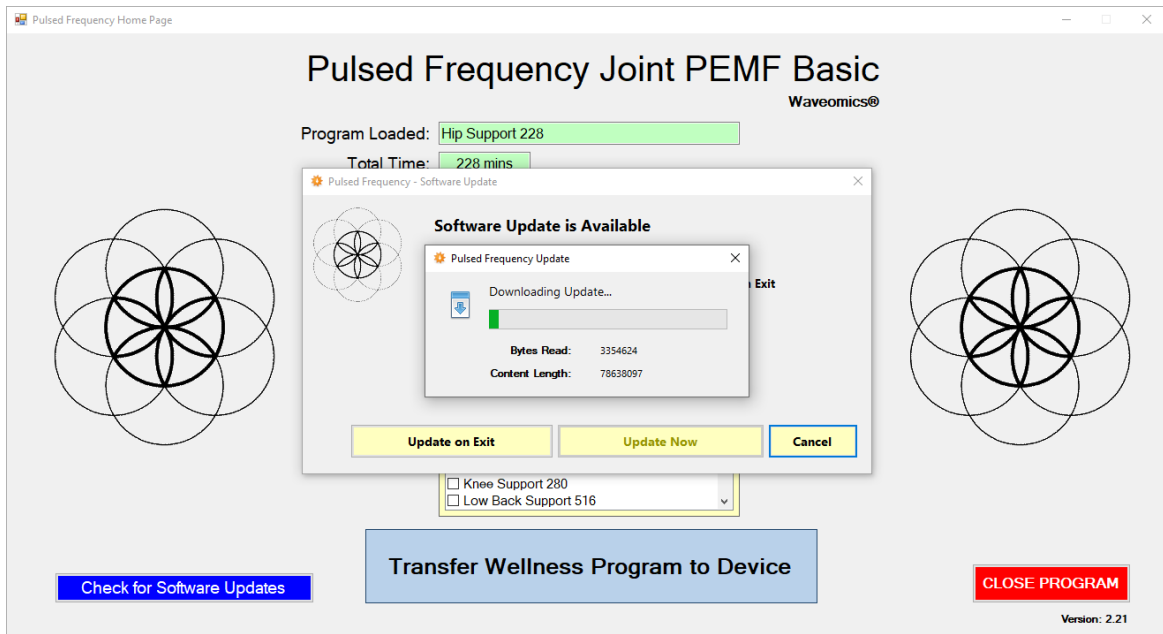


If you clicked **Update Now**, a progress bar will be displayed as the software is being downloaded.

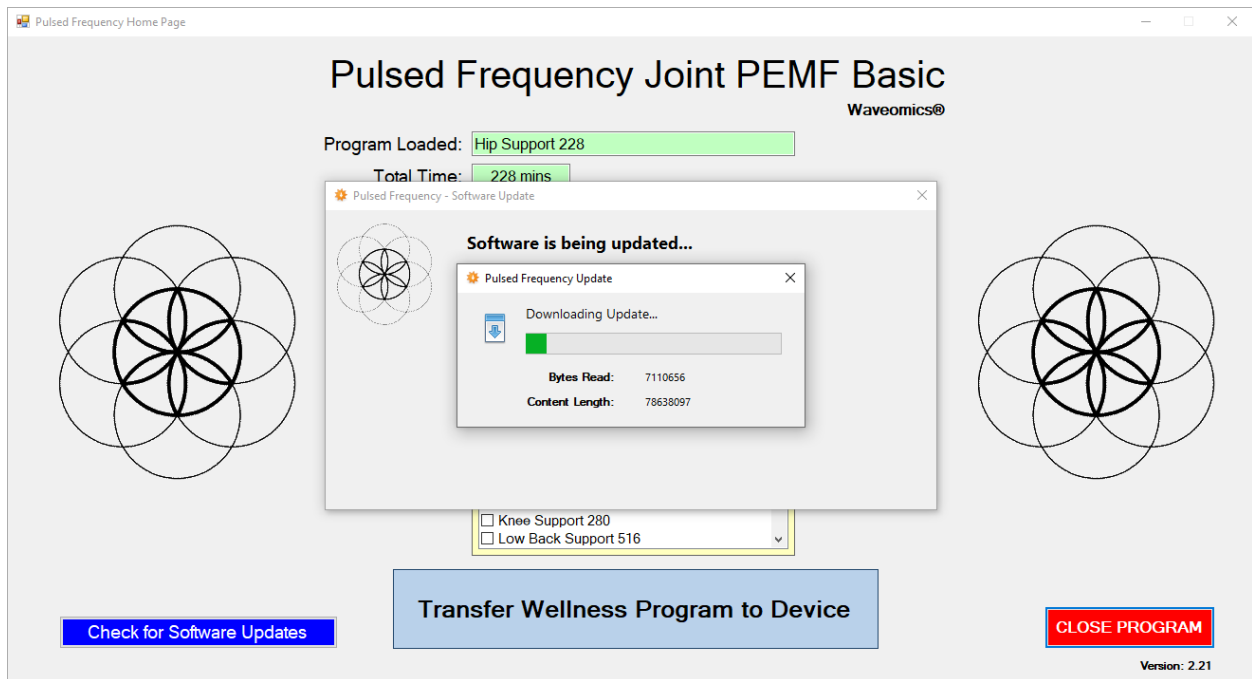

If you clicked **Update on Exit**, the above dialog box will be displayed and the update downloaded when you click the **Close Program** button.


Load a Program into the Joint PEMF

Before running the JOINT PEMF Basic software, make sure your JOINT PEMF is connected to the computer.

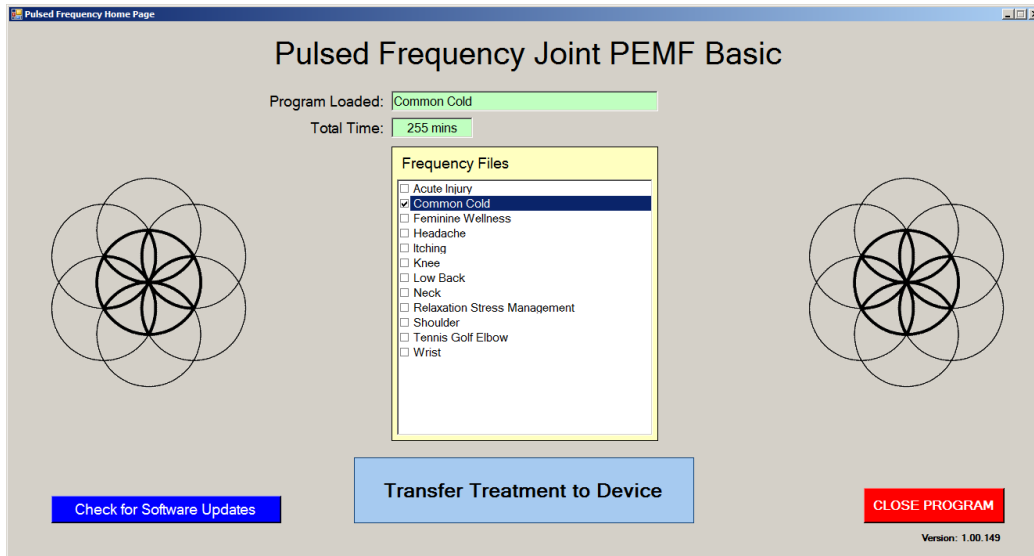

The JOINT PEMF Basic software is very easy to use. Select one of the available Wellness programs from the list. When you do this, the program name and amount of time it runs for will be displayed at the top.

When you are ready, click the **Transfer Treatment to Device** button. The Wellness program will then be transferred to the JOINT PEMF.

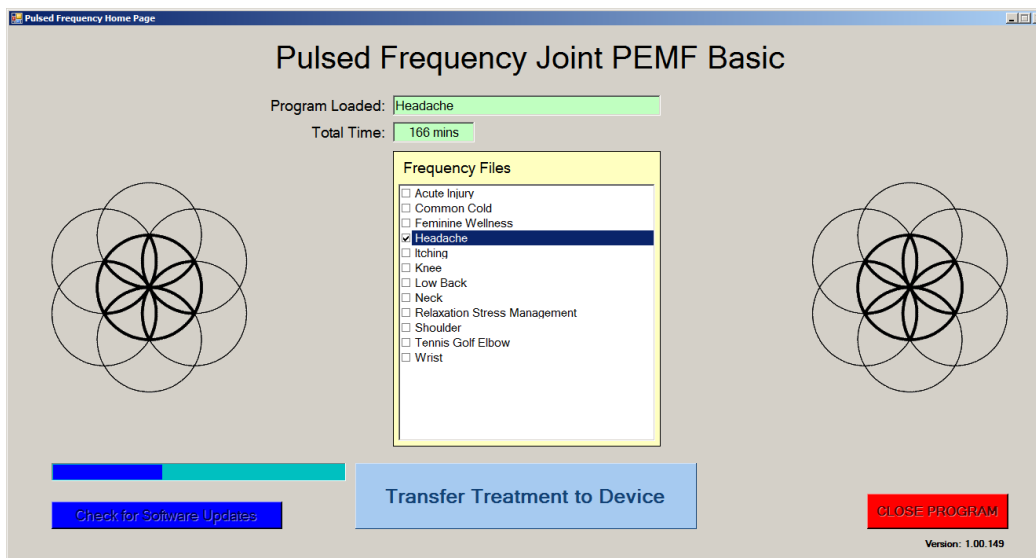

As the program is being transferred, you will see a progress bar in the lower left hand corner. After the transfer has finished, the JOINT PEMF will be ready to use.

Troubleshooting

If you have problems detecting your JOINT PEMF device, you will need to troubleshoot the problem. To do this, exit from the software and try the following solutions.

Solution 1

If the JOINT PEMF is turned on when you connect it to the computer and run the software, you won't be able to do anything. The normal procedure is to turn the JOINT PEMF off, connect it to the computer, then run the software. If you do that, everything will work properly.

Solution 2

Even though you have the USB cable plugged into the JOINT PEMF and to the computer, you may not have a good connection. Unplug the cable from both the JOINT PEMF and the computer, then plug the cable back into both places. Next, run the software, click View Frequency Files in Device, and see if the words, "JOINT PEMF is not connected" still appear on the screen. If they don't, your problem is solved. You can now proceed to the Software User Guide section. If you still see the words, proceed to Solution 2.

Solution 3

The most common cause of a connection problem is that Windows cannot see your device. Try plugging the USB cable into a different USB port on your computer. Also, unplug the cable from your JOINT PEMF and plug it back in. Run the software again, click View Frequency Files in Device and see if you can see the files loaded into your JOINT PEMF. If you can see the files, proceed to the Software User Guide section. If you still get the message, "JOINT PEMF is not connected", proceed to Solution 3.

Solution 4


If you are still having a problem, try plugging the larger end of the USB cable into a USB hub. The hub must be powered for this to work. A powered USB hub is where a power cable runs from the hub to an AC adapter that plugs into a wall outlet. With the JOINT PEMF plugged into the hub, run the software and try to load a treatment. If you can, your problem is solved.

Solution 5

Another cause of a connection problem is a bad USB cable. If you suspect your cable, then replace it and try again.

Solution 6


If all other solutions have failed, a final check is to run the Device Manager.

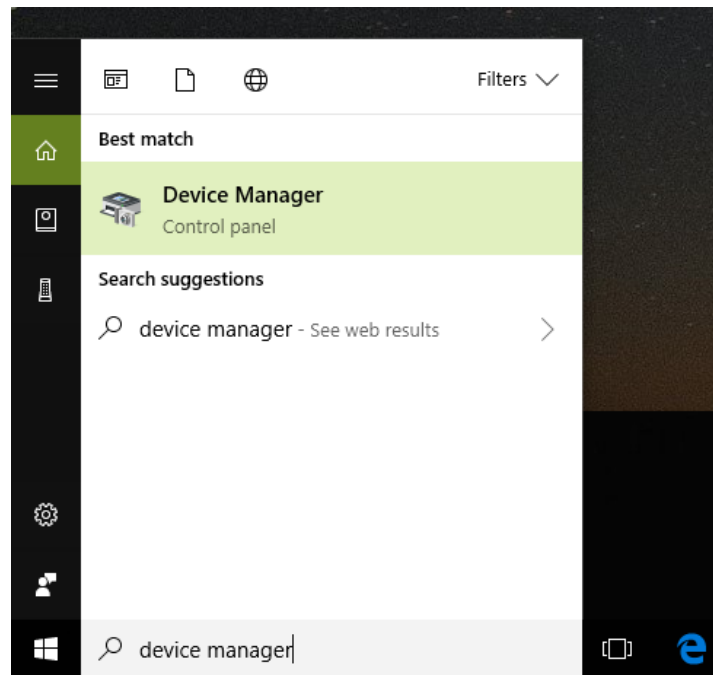

If you are using Windows 10, go to the lower left hand corner of the screen and enter the words "device manager". When that choice appears in the list, click on it.


If you are using Windows 8 or 8.1, click the flag in the lower left hand corner and type "device manager." When that choice appears in the list, click on it.

If you are on Windows 7, click the Start button in the lower left hand corner, and enter "device manger" in the search box. When that choice appears in the list, click on it.

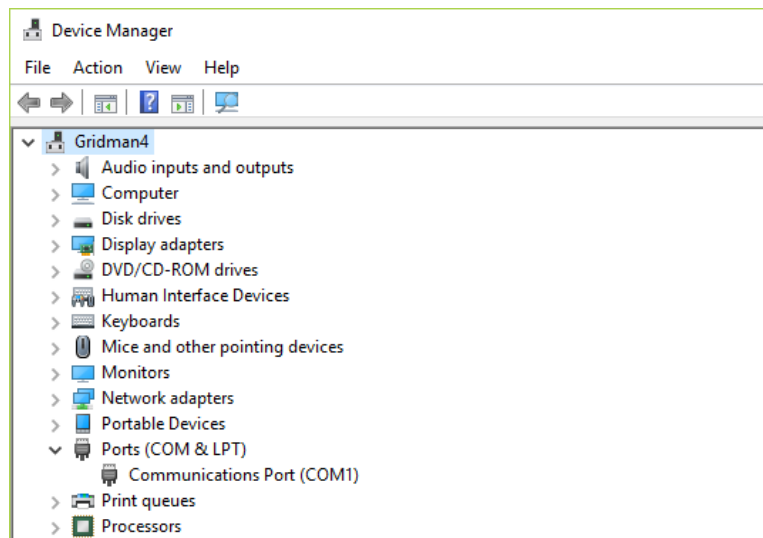

In Device Manager, click the arrow next to Ports (COM & LPT) in order to expand the list. If the JOINT PEMF is connected and you see something similar to the screenshot above, then

Windows cannot see your device. Try any of the above solutions then come back to Device Manager and look at the list.

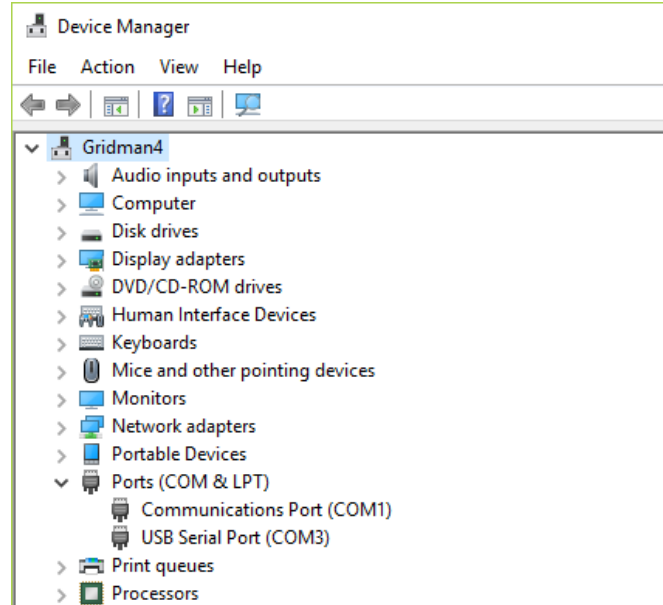

When Windows can see your JOINT PEMF, you will see an entry similar to the one above that says, "USB Serial Port (COM3)". It may show a different COM port, but that is okay. The main thing to remember is that when Windows can see your JOINT PEMF, it will assign a COM port to it. You should now be able to run the software and transfer treatments to the JOINT PEMF.

Joint PEMF Manual Detection


Please follow these steps in order to manually detect your JOINT PEMF.

On Windows 10, click the Windows flag in the lower left corner of the screen and scroll down the list of programs until you see the Pulsed Frequency folder. Expand that folder and select **Detect JOINT PEMF**.


On Windows 8 or 8.1, click the Windows flag in the lower left corner of the screen, type "detect Joint PEMF" and when that item is displayed in the list on the right, select it to run the utility.

On Windows 7, go to the Start Menu, click All Programs, scroll down to the Pulsed Frequency folder and select the **Detect JOINT PEMF** program. Make sure your JOINT PEMF is not connected to the computer when you do this.


After you have clicked on the shortcut, you will get a warning from Windows saying "Do you want to allow this app from an unknown publisher to make changes to your device?" Click "Yes". Follow the instructions on the screen to detect your device.

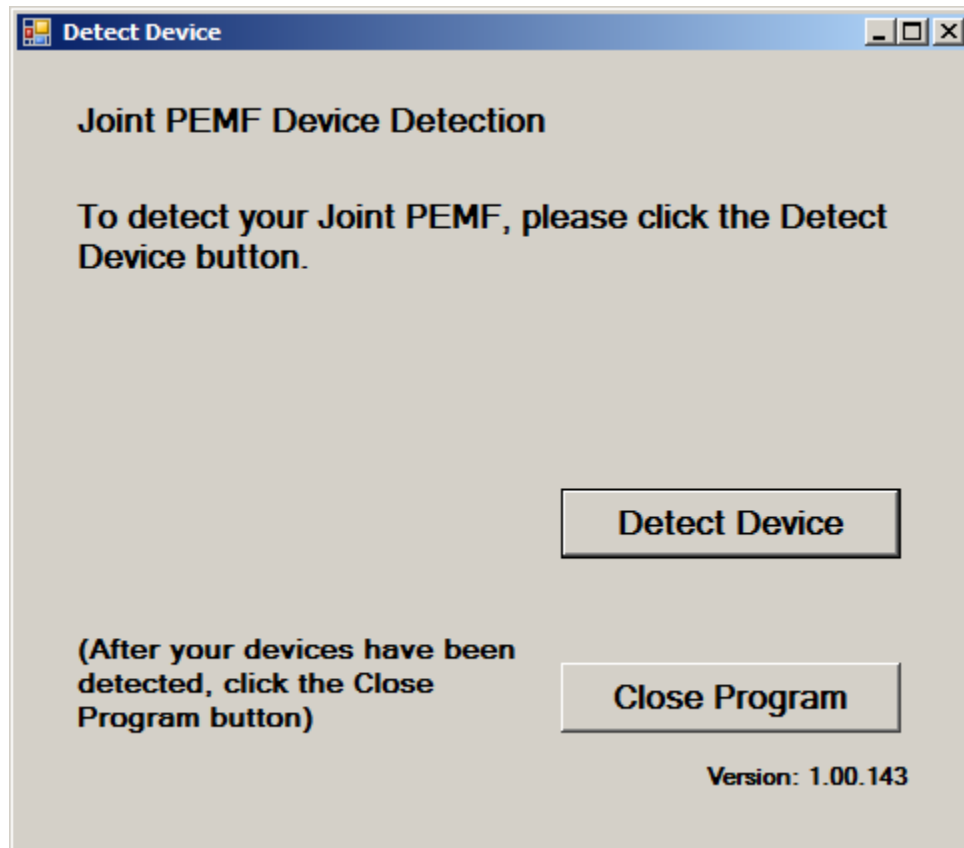


On the first page of the utility program, click the Detect Device button. The following dialog box will then be displayed.

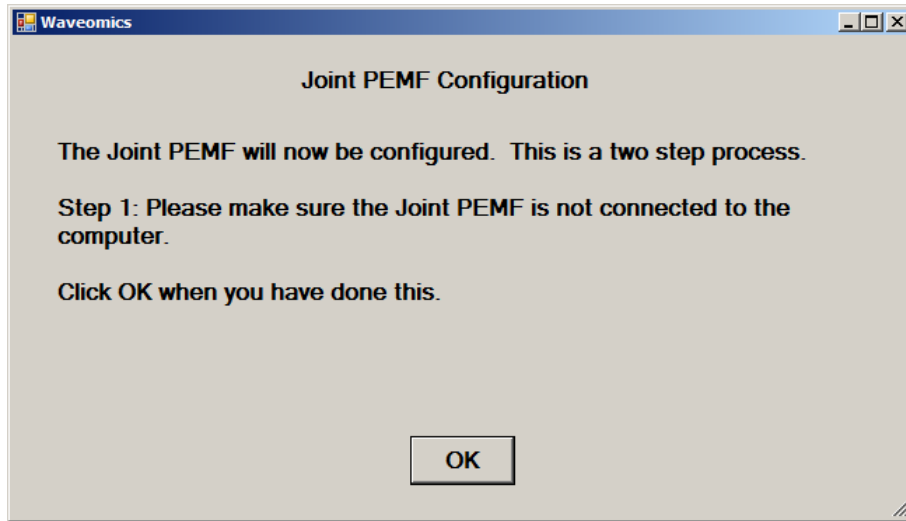


Make sure the JOINT PEMF is not connected to the computer, then click the **OK** button.

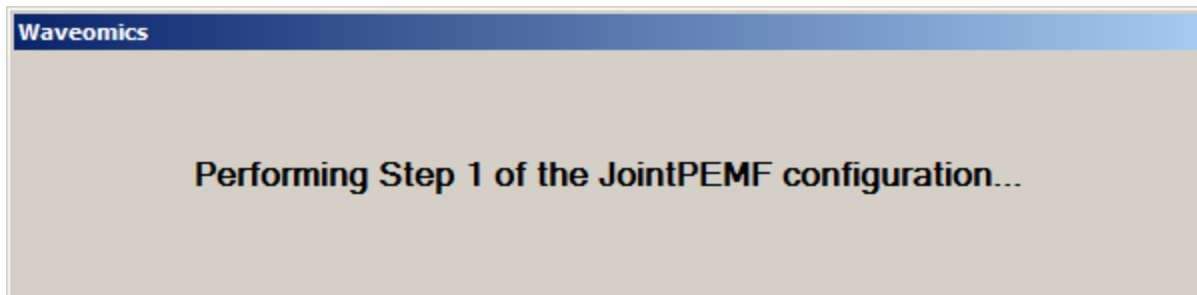


Here, the program is looking at your system without the JOINT PEMF connected. After it is finished, the following dialog box will be displayed.

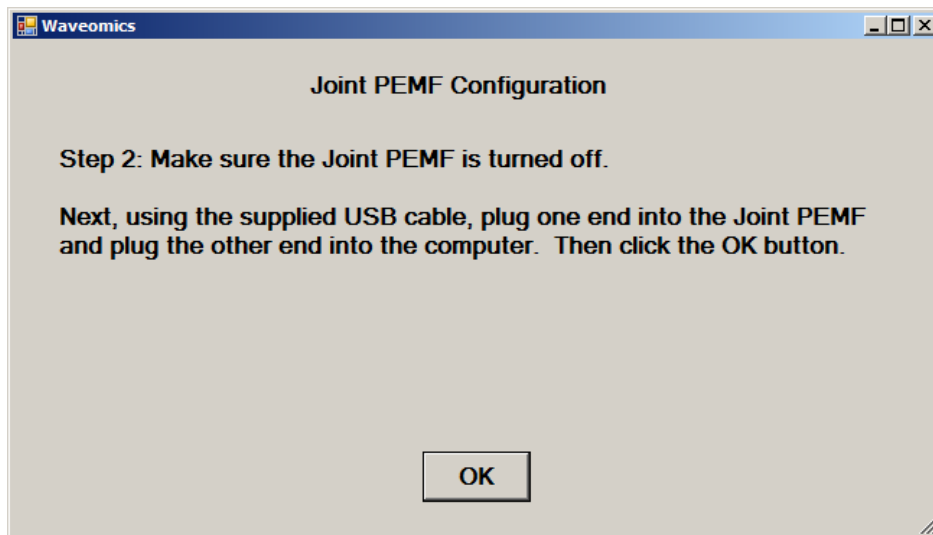


Make sure the JOINT PEMF is turned off, connect it to the computer, then click the **OK** button.

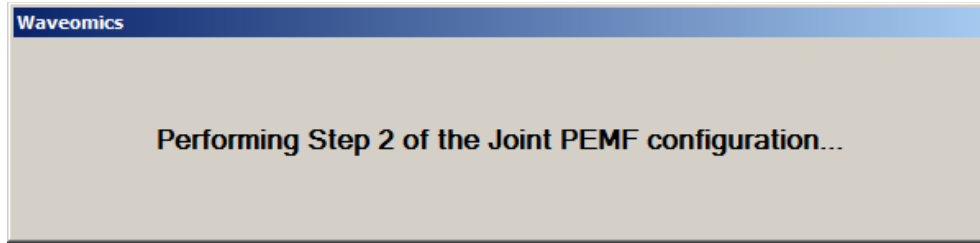

Here, the utility program is looking at your system with the JOINT PEMF connected.

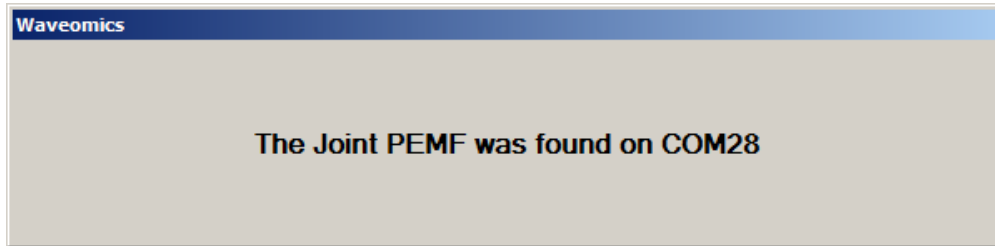

When it has detected your device, you will be returned to the main screen of the program.

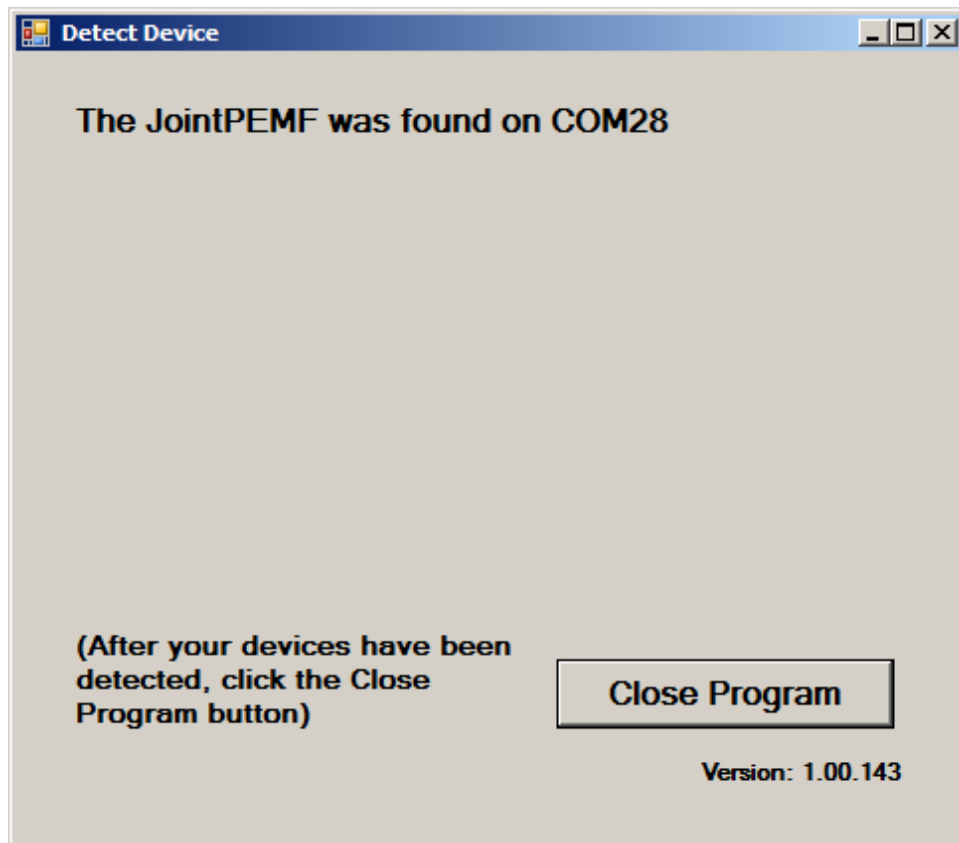

The location of the JOINT PEMF will be displayed on the screen and saved in the configuration information for the Pulsed Frequency JOINT PEMF Basic software. You can now click the **Close Program** button to exit the program. This completes the manual detection process.

The Pulsed Frequency™ software can now be run with the JOINT PEMF. Go to the desktop and double-click on the Joint PEMF Basic shortcut.

