Pulsed Frequency™



MagRez™ B Basic Software Download, Installation and User Guide

Contents

Account Registration	3
Create an Account	4
Download the Software	6
Chrome	7
Edge	8
Firefox	10
Install the Software	12
MagRez B Pulsed Frequency Generator	17
MagRez™ B Operation	17
Charging the MagRez™ B	18
Placement of the MagRez™ B	19
Connect the MagRez™ B to Your Computer	20
When the Software is Run	21
Load a Program into the MagRez	23
Troubleshooting	24
MagRez™ B Manual Detection	27

Account Registration

To download the Pulsed Frequency software, please go to www.pulsedfrequency.com.



Click where the arrow is pointing. You will be taken to a Login screen.

Pulsed Freque	ncy								
	HOME	REGISTER	SOFTWARE	FREQUENCY LISTS	RESOURCES	FAQ	LO		
Login									
Login									
C									
You are not logged in.									
	I hav	/e an acco	unt I war	nt to create an a	ccount				
E-mail *									
Decoverd *									
Passworu "									

If you already have an account, click **I have an account**. Enter your email address and password and then click **Log in**.

Create an Account

If you do not have an account, select I want to create an account.

Pulsed Freque	ency							
	HOME	REGISTER	SOFTWARE	FREQUENCY LISTS	RESOURCES	FAO	LO	
Login								
Login								
0								
You are not logged in.								
				00				
	I hav	e an acco	unt I wa r	nt to create an a	ccount			
Please check your	spam folde	er for the confi	rmation email.					
E mail *								
E-IIIdii "								
E-mail *								
Create new accou	unt							

Enter your email address, then click **Create new account**. Further instructions will then be sent to your email address. The confirmation email will look something like this.

From: To: Cc: Subject:	support@pulsedfrequency.com rod@server.com Account details for rod 564 at Pulsed Frequency							
Dear R	egistered User,							
Thank this l	you for registering at Pulsed Frequency. You may now log in by clicking ink or copying and pasting it to your browser:							
<u>https:</u>	//pulsedfrequency.com/user/reset/564/1534971011/JKZCwxsqdPQhoLdXsoli2wnP8Z5S4uiCxLi-LJuTjH0							
This l you ca	ink can only be used once to log in and will lead you to a page where n set your password.							
After <u>https:</u>	After setting your password, you will be able to log in at https://pulsedfrequency.com/user in the future using:							
userna passwo	ne: rod@server.com rd: Your password							
Pu	lsed Frequency team							

Click on the link in the email and you will be taken back to the website to reset your password.

You ha	ve just used your one-time login link. It is no longer necessary to use this link to log in. Please change your password.	×
View	Edit Orders	
-mail a	ddress *	
rod@s	server.com	
Passw	ord	
Confir	m password	
Email	updates	
☑ Subs	scribe	
Locale	e settings	
Time z	one	
Ame	rica/New York: Wednesday, August 22, 2018 - 5:06pm -0400	•

Enter a New password in both fields.

You have just (used your one-time login link. It is no longer necessary to use this link to log in. Please change your password.	×
View Edit	Orders	
-mail address	*	
rod@server.co	m	
Password	•	
Confirm passv	vord	
Email update	es	
Subscribe		
Locale settin	gs	
Time zone		
America/Nev	v York: Wednesday, August 22, 2018 - 5:06pm -0400	•
✓ Save		

After you have changed your password, click the **Save** button. You will be told that the changes have been saved.

Download the Software

If you aren't already logged in, log into your account, then click the Software menu item.



Point to **Download**, point to **Basic**, then click **MagRez Basic**. When you do this, you will be taken to the following page.



Click the **Download** button.

What happens next depends on the web browser you are using. Here are instructions specific to the three main browsers being used today.

Chrome



When you click the **Download** button, a notification displays in the lower left hand corner of the browser that alerts you to the file being downloaded. You will see the number of bytes that have been downloaded and the total size of the file.

	Pulsed Free	quency	HOME	SOFTWARE	FREQUENCY LISTS	RESOURCES	FAQ	LOGOUT
	WAVEOMICS™	Software: Pulse	d Frequen	cy™ MagRez Bas	ic			
	WAVE MagR	OMICS ez Bas	S™ S ic	Softwa	re: Pulsed	Freque	enc	У ^{тм}
	View Edit	Access contro	Devel					
	<i>You must be log</i> Download	gged in to downl	oad Waved	mics™ software.				
		Open when done Always open files of	this type					
		Pause Show in folder						
P	ulsedFrequencyEexe	Cancel						Show all

When the file download has completed, click the down arrow and then click **Open**. This will run the installation program for the **MagRez Basic** software.

Edge

When you click the **Download** button, a small rectangular window will display at the bottom of the screen.



This window will show the progress of the file being downloaded.

🛞 Pulsed Frequer	псу						
	HOME	SOFTWARE	FREQUENCY LISTS	RESOURCES	FAQ	LOGOUT	
WAVEOMICS™ Soft	tware: Pulse	ed Frequency™ M	lagRez Basic				
WAVEOMICS™ Software: Pulsed Frequency™ MagRez Basic							
View Edit A	ccess contro	ol Devel					
You must be logged	d in to down	nload Waveomic	s™ software.				
Download							
The publisher of PulsedFre	≥quencyEM27.ex	xe couldn't be verified.	Are you sure you want to run the	e Dun	Migur d	laumlaada V	

When file download has completed, click the **View Downloads** button.

Add co	bard Context Structure Store Appearance People Hodd Atent Find content	ins Configuration Reports Help			☆ ≋ ⊙ <u>∔</u>	#
-02					DOWNLOADS	Open folder
8	Pulsed Frequency	HOME SOFTWAR	RE FREQUENCY LISTS	RESOUR	Past Downloads PulsedErectuercoMacEezBasic ex	Clear all
	Pulsed Frequency [™] MagRez B	Basic			pulsed/requency.com This program is not commonly downloaded an	×
	Pulsed Freque	ncy™ Ma	gRez Basic	Sof	could harm your computer.	
	View Edit Revisions Acces	ss control Devel				
	You must have an account and be k	ogged in to download s	oftware.			

In the upper right hand corner of the screen above, click **Open Folder**.

🖊 🛃 📙 🖛 Do	wnload	ls			_	\times
File Home	Share	View				× 🕐
	> Th	is PC > Downloads		ٽ ~	Search Downloads	P
📌 Quick access		Name	Date modified	Туре	Size	
E Desktop	*	🔩 PulsedFrequencyMR7Basic.exe	1/18/2019 7:02 AM	Application	76,878 KB	
👆 Downloads	*					
Documents	*					
Pictures	*					
👌 Music						
Videos						

Double-click **PulsedFrequencyMagRezBasic.exe** to run the installation program for the **MagRez Basic** software.

Firefox

When you clicked the **Download** button, the following dialog box popped up.

Opening PulsedFrequencyMagRezBasic.exe		×
You have chosen to open:		
PulsedFrequencyMagRezBasic.exe		
which is: Binary File		
from: https://dl.boxcloud.com		
Would you like to save this file?		
	Save File	Cancel

In this dialog box, click the Save File button.



Next, go to the upper right hand corner of the browser and click the **Down Arrow**.

Pulsed Frequency



A small window will appear. Click the link in the lower left that says, Show All Downloads.



Next, the above window will pop up where you can watch the progress of the downloading file. When the file has downloaded, double-click on PulsedFrequency MagRez Basic.exe and the installation program will run.

Install the Software

To install the software, double-click on the downloaded file.



On this Welcome screen, click the Next button.



Click I accept the terms of the license agreement, then click the Next button.

×

To begin the software installation, click the **Install** button.



At this point, this dialog box will appear and inform you that the drivers needed for your MagRez will now be installed. Click the **OK** button to begin.



On this screen click the Next button.

Device Driver Installation Wizard			
License Ag	reement		
Ń	To continue, accept the following license agreement. To read the entire agreement, use the scroll bar or press the Page Down key.		
	IMPORTANT NOTICE: PLEASE READ CAREFULLY BEFORE INSTALLING THE RELEVANT SOFTWARE: This licence agreement (Licence) is a legal agreement between you (Licensee or you) and Future Technology Devices International Limited of 2 Seaward Place, Centurion Business Park, Glasgow G41 1HH, Scotland (UK Company Number SC136640) (Licensor or we) for use of driver software provided by the Licensor(Software).		
BY INSTALLING OR USING THIS SOFTWARE YOU AGREE TO THE \checkmark			
	O I accept this agreement Save As Print		
	< Back Next > Cancel		

Accept the license agreement, then click the **Next** button.



Click the **Finish** button to complete the driver installation.



At this point, the MagRez Basic software will be installed. Just wait until it has completed.



When you see this screen, the installation is complete. Click the **Finish** button.



During installation, a shortcut to the **MagRez Basic** software was placed on your desktop. You can double-click this shortcut to run the software.

MagRez B Pulsed Frequency Generator

The **MagRezTM B Pulsed Frequency Generator** comes with a frequency generator disc, a 110-volt AC voltage adapter, USB power/programming cord, carrying case, and 1 pack of NikStix.



The MagRezTM B Kit contents

MagRezTM B Operation

To turn on the **MagRezTM B**, press the green ON button, located on the bottom panel. When the **MagRezTM B** is operational the GREEN RUN LED in the lower left corner of the top face of the **MagRezTM B** flashes.

The **MagRezTM B** has an internal lithium ion rechargeable battery. When the battery is discharged, the unit will turn itself off until it has been recharged. During normal use it will run programmed treatments for at least 15 hours between charges. For longer continuous operation, the unit may run treatments with the charger connected. The unit cannot be damaged by overcharging, and it is recommended to charge the unit when not being used to keep the batteries near full charge.

When the unit is being charged, the LED shows RED. When the charge is complete, the LED shows GREEN. If the unit is running while charging, the LED will alternate between RED/ORANGE colors. The LED is not illuminated if it is not running nor being charged.



MagRezTM B after being turned on

Charging the MagRezTM B



MagRez[™] B side view

Insert the small USB connector into the MagRezTM B charging/programming port located to the right of the On/Off button. Notice that one side of the USB connector is narrower than the other. Turn the USB connector so that the narrow side is on the bottom.

The larger USB connector plugs into the 110-volt AC voltage adapter. There is a white strip in the female receptacle on the AC adapter, and another white strip in the male USB connector. The USB connector only fits into the adapter one way, when the two white strips are on opposite sides.



When the **MagRezTM B** is operational the GREEN RUN LED in the lower left corner of the top face of the **MagRezTM B** flashes.

Placement of the MagRezTM B

Place the **MagRezTM B** anywhere on your body where there is pain or injury. It conveniently fits in any pocket. The side with the blue Flower of Life design should face away from the body. You can use NikStix double-sided adhesive disks to adhere the **MagRezTM B** to your body. The adhesive lasts for 24 hours. The disks can be purchased on the internet at NikStix.com.

Connect the MagRezTM B to Your Computer

In order to program the MagRezTM B device, you must connect it to your computer.



To do this, take the large end of the USB cable that is attached to your device and plug it into a USB port on your computer.



To run the software, double-click the MagRez[™] shortcut on your desktop.

When the Software is Run

When you run the software, it will first check to see if a software update is available.

	– 🗆 X
sed Frequency MagRez Basic Waveomics®	C
.oaded: Wrist Support 224	
al Time: 224 mins	
Wellness Programs	
Acute Injury 452 Back Pain 353 Bone Injury 473 Pulsed Frequency − □ ×	
Checking for updates, please wait	
Gastrointestinal Support 507 Headache 166 Heart Support 326 Hip Support 228 Insect Bites 191 Iching 40 Knee Support 280 Low Back Support 516 v	
Transfer Wellness Program to Device	CLOSE PROGRAM
	sed Frequency MagRez Basi waveonics® oaded: Wrist Support 224 al Time: 224 mins <u>Wellness Programs</u> <u>Oacute Injury 452</u> <u>Back Pain 353</u> <u>Bone Injury 473</u> <u>Puted Frequency</u> <u>Checking for updates, please wait</u> <u>Gastrointestinal Support 507</u> <u>Headache 166</u> <u>Heart Support 228</u> <u>In sect Bites 191</u> <u>In sect </u>

During this process, a small dialog box will display. If no update is available, the dialog will just go away.

🎄 Pulse	d Frequency - Software Update		×
	Software Update is A To install available updates r To install available updates u	vailable now, click the Update button. upon exit, click the Update on Ex	it
	Installed Software Version: Available Software Version:	2.38.000 2.39.000	
	Update on Exit	Update Now	Cancel

If an update is available, the above dialog box will appear. If you click **Update on Exit,** this box will disappear and you will be back at the main page. The software will then be updated when you click the **Close Program** button. If you click **Update Now**, the software will be updated immediately. If you click **Cancel**, no update will be performed.

📓 Pulsed Frequency Home Page	– 🗆 X
Pulsed Frequency MagRez Basic	
Program Loaded: Wrist Support 224	
Total Time: 224 mins Pulsed Frequency - Software Update X	
Software Update is Available Putred Frequency Update Pownloading Update Putred Frequency Update Pownloading Update Prover Read: 8200192 Cortent Length: 78748475 Update One Exit Update Now Cancel	
Wrist Support 224	
Check for Software Updates	CLOSE PROGRAM
	Version: 2.38

If you clicked **Update Now**, a progress bar will be displayed as the software is being downloaded.

😼 Pulsed Frequency Home Page	- 🗆 X
Pulsed Frequency MagRez Bas	ic
Program Loaded: Wrist Support 224	
Total Time: 224 mins	
🔅 Pulsed Frequency - Software Update	×
Software is being updated Version of the second	
□ Workout Recovery 197 ✓ Wrist Support 224 ✓	
Check for Software Updates	CLOSE PROGRAM
	Version: 2.38

If you clicked **Update on Exit**, the above dialog box will be displayed and the update downloaded when you click the **Close Program** button.

Load a Program into the MagRez

Before running the **MagRez Basic** software, make sure your **MagRezTM B** is connected to the computer.

Rulsed Frequency Home Page		- 🗆 X
Pulsed	Frequency MagRez	Basic omics®
Program Loaded:	Wrist Support 224	
Total Time:	224 mins	
	Wellness Programs	
	Acute Injury 452 Back Pain 353 Bone Injury 473 Oommon Cold 255 Elbow Support 210 Emotional Support 196 Featgue 196 Featgue 196 Gene Better 252 Feminine Wellness 180 Gastrointestinal Support 507 Headache 166 Heart Support 326 Hip Support 228 Insect Bites 191 Iching 40 Knee Support 280 Low Back Support 516 ✓	
Check for Software Updates	sfer Wellness Program to De	VICE CLOSE PROGRAM
		Version: 2.38

The **MagRez Basic** software is very easy to use. Select one of the available Wellness programs from the list. When you do this, the program name and amount of time it runs for will be displayed at the top.

When you are ready, click the **Transfer Treatment to Device** button. The Wellness program will then be transferred to the $MagRez^{TM} B$.



As the program is being transferred, you will see a progress bar at the lower left hand corner of the screen. When the program has finished loading, the $MagRez^{TM} B$ will be ready to use.

Troubleshooting

If you have problems detecting your $MagRez^{TM} B$ device, you will need to troubleshoot the problem. To do this, exit from the software and try the following solutions.

Solution 1

The **MagRezTM B** must be turned off before you connect it to the computer. It does not matter if the software is running or not when you connect the device. The key is that the **MagRezTM B** must be turned off when you connect the USB cable to it.

Solution 2

Even though you have the USB cable plugged into the **MagRezTM B** and to the computer, you may not have a good connection. Unplug the cable from both the **MagRezTM B** and the computer, then plug the cable back into both places. Next, run the software, click View Frequency Files in Device, and see if the words, "MagRez is not connected" still appear on the screen. If they don't, your problem is solved. You can now proceed to the Software User Guide section. If you still see the words, proceed to Solution 2.

Solution 3

The most common cause of a connection problem is that Windows cannot see your device. Try plugging the USB cable into a different USB port on your computer. Also, unplug the cable from your **MagRezTM B** and plug it back in. Run the software again, click View Frequency Files in Device and see if you can see the files loaded into your **MagRezTM B**. If you can see the files, proceed to the Software User Guide section. If you still get the message, "MagRez is not connected", proceed to Solution 3.

Solution 4

Sometimes, the **MagRezTM B** can be in an error state, and no amount of plugging and unplugging cables will solve the problem. To test if your device is in an error state, press the power button momentarily, and then immediately let it up. If you hear a beep, the device was in an error state, and you just cleared the error. Run the software again and try to load a treatment onto the **MagRezTM B**. If you are able to, your problem is solved.

Solution 5

If you are still having a problem, try plugging the larger end of the USB cable into a USB hub. The hub must be powered for this to work. A powered USB hub is where a power cable runs from the hub to an AC adapter that plugs into a wall outlet. With the **MagRezTM B** plugged into the hub, run the software and try to load a treatment. If you can, your problem is solved.

Solution 6

Another cause of a connection problem is a bad USB cable. If you suspect your cable, then replace it and try again.

Solution 7

If all other solutions have failed, a final check is to run the Device Manager.



If you are using Windows 10, go to the lower left hand corner of the screen and enter the words "device manager". When that choice appears in the list, click on it.

If you are using Windows 8 or 8.1, click the flag in the lower left hand corner and type "device manager." When that choice appears in the list, click on it.

If you are on Windows 7, click the Start button in the lower left hand corner, and enter "device manger" in the search box. When that choice appears in the list, click on it.



In Device Manager, click the arrow next to Ports (COM & LPT) in order to expand the list. If the **MagRezTM B** is connected and you see something similar to the screenshot above, then Windows cannot see your device. Try any of the above solutions then come back to Device Manager and look at the list.



When Windows can communicate with your $MagRez^{TM} B$, you will see an entry similar to the one above that says, "USB Serial Port (COM3). It may show a different COM port, but that is okay. The main thing to remember is that when Windows can see your $MagRez^{TM} B$, it will assign a COM port to it. You should now be able to run the software and transfer treatments to the $MagRez^{TM} B$.

MagRezTM B Manual Detection

Please follow these steps in order to manually detect your MagRezTM B.

On Windows 10, click the Windows flag in the lower left corner of the screen and scroll down the list of programs until you see the Pulsed Frequency folder. Expand that folder and select **Detect MagRez**.

On Windows 8 or 8.1, click the Windows flag in the lower left corner of the screen, type "detect magrez" and when that item is displayed in the list on the right, select it to run the utility.

On Windows 7, go to the Start Menu, click All Programs, scroll down to the Pulsed Frequency folder and select the **Detect MagRez** program. Make sure your **MagRezTM B** is not connected to the computer when you do this.

After you have clicked on the shortcut, you will get a warning from Windows saying "Do you want to allow this app from an unknown publisher to make changes to your device?" Click "Yes". Follow the instructions on the screen to detect your device.



On the first page of the utility program, click the **Detect Device** button. The following dialog box will then be displayed.



Make sure the MagRez[™] B is not connected to the computer, then click the OK button.

Waveomics		
	Performing Step 1 of the MagRez configuration	

Here, the utility program is looking at your system without the $MagRez^{TM} B$ connected. After this is finished, the following dialog box will be displayed.



Now, connect the MagRezTM B to the computer, then click the OK button.

Waveomics

Performing Step 2 of the MagRez configuration...

Here, the program is looking at your system with the MagRezTM B connected.

Waveomics		
	The MagRez was found on COM26	

When your device has been detected, you will be returned to the main screen of the program.

💀 Detect Device	- 0	×
The MagRez was found on CO	M26	
(After your devices have been detected, click the Close Program button)	Close Program	
	Version: 2	2.40

The location of the **MagRezTM B** will be displayed on this screen and its location will have been saved in the configuration information for the **Pulsed Frequency MagRezTM B** software. You can now click the **Close Program** button to exit the utility. This completes the manual detection process.

The **MagRez Basic** software can now be run with the **MagRez[™] B**. Go to the desktop and double-click on the **MagRez Basic** shortcut.

